

THE AGE OF RADIOTHERAPY  
**STERITATT**

**Ultra Violet Pink Ink Tips:**

Keep in mind the UV Pink ink is less vicious than the Black ink and is a bit more challenging to use.

Shake UV cartridges well prior to use

Attach large gauge needle, 18g -20g. Squeeze Ink Bulb to and start a nice controlled flow of ink through needle lumen.

**The Dropping technique is not recommended for UV inks.** The Vaccination technique is more appropriate and will ensure getting the ink under the skin.

When first using the UV ink it is important to get used to its performance so we always suggest using on “INSIGNIFICANT POINTS”, superior border, a corner mark etc. After a few procedures on various anatomical areas you will become comfortable with the viscosity and the adherence to the skin. Once the end users are comfortable they can move on to using UV ink as the main ink for ISO and all points. This is good practice to avoid any issues.

Keep in mind when using the larger gauge needle the UV component in the ink sometimes takes a few days before you can see it shimmer. This is due to the skin trauma overshadowing the mark.

Please feel free to contact me with any questions.

Thank you

James Matera

[james@steritatt.com](mailto:james@steritatt.com)