

THE AGE OF RADIOTHERAPY

STERiTATT

PATENTED MEDICAL MARKING DEVICES

Sterile Tattooing for Radiation Delivery

SteriTatt is a single-use device that contains sterile, non-toxic, permanent ink. Each SteriTatt device contains ink designed to reduce skin sensitivity that is safety-sealed and gamma irradiated, guaranteeing sterility and ensuring the highest quality of patient care. SteriTatt's pen-like design offers ease and comfort of use and helps standardize the methods therapists employ to administer tattoos. The ink flow is easy to regulate so too much ink will not be dispersed into a patient's skin – controlling the cosmetic impact a tattoo will have in the patient.

SteriTatt offers black and green ink. Different ink colors can be used to identify different areas of treatment or differentiate new areas from previous treatment sites. The markings made are permanent and are very small. The SteriTatt tattoo enables providers to align the treatment field precisely, every day of the treatment to ensure that radiation is delivered exactly to the same area each time.

Features:

- Sterile
- Non-toxic, designed to reduce skin sensitivity
- Single Use for Patient Safety
- Neat and Easy to Use
- Standardizes Tattooing Method
- Supports Precision of Treatment

THE AGE OF RADIOTHERAPY


STERiTATT

PATENTED MEDICAL MARKING DEVICES

INSTRUCTIONS FOR USE


Follow current department protocols regarding intradermal procedures.

VACCINATION TECHNIQUE

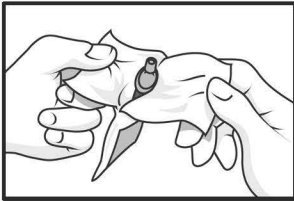


Verify that SteriTatt package is completely sealed and unopened.


Remove SteriTatt device from packaging. Hold down on cap and shake firmly.

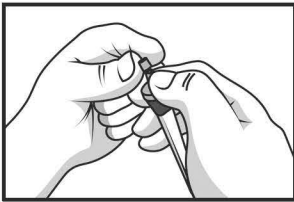


Remove and discard plastic cap.

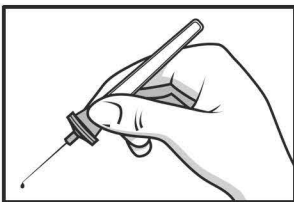

Attach needle to SteriTatt device. Best results are achieved with a Short Stem 21 GA 1" needle. Hold SteriTatt at ink bulb between thumb and index finger. Over trash receptacle, squeeze to produce ink drops from needle tip.

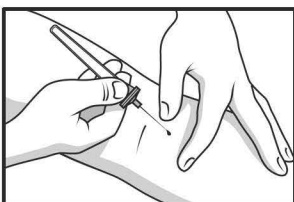

With free hand, hold patient's skin taught over tattoo point. Gently insert needle point (lumen) under skin surface. Carefully squeeze ink bulb, visualize ink collection at skin surface, and remove needle. *

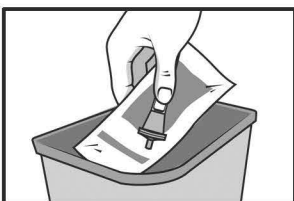


Dispose SteriTatt product in normal trash.

*For a slightly larger tattoo, gently twist needle after introducing ink.

Cleaning Instructions: Use soap and water to wipe up spills.


Rx Statement : United States Federal law restricts this device to sale by or on the order of a physician

DROPPING TECHNIQUE

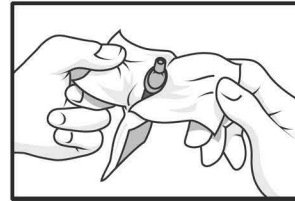


Verify that SteriTatt package is completely sealed and unopened.


Remove SteriTatt device from packaging. Hold down on cap and shake firmly.

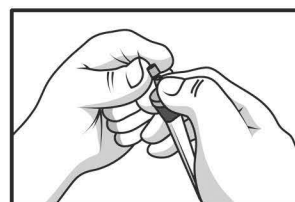


Remove and discard plastic cap.

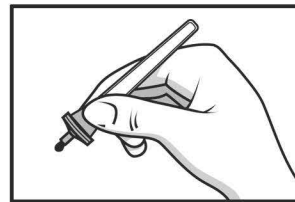

Hold SteriTatt at ink bulb between thumb and index finger. Over trash receptacle, squeeze to produce ink drops from tip.

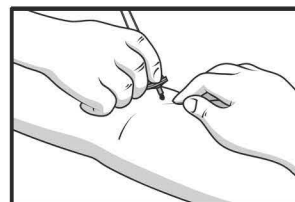

Gently insert desired needle point (lumen) under skin surface. Carefully squeeze ink bulb, dripping ink onto needle tip. Twist needle and remove.

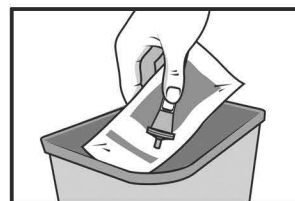

Dispose needle as per hospital/facility regulations. Dispose SteriTatt product in normal trash.



Date of Manufacture Symbol



Use By Symbol